

### **Submittal Data Sheet**

#### **Features and Benefits**

The Tri-Tech Med Touch Area Alarm Panel digitally displays gas pressure (1 psi increments) and provides alarm conditions as required by the latest edition of NFPA 99 for up to 16 medical gases. The alarm is ETL listed to UL 1069 and CSA C22.2 # 205 Signal equipment. Transducers are included. DISS risers are included - transducers may be installed remotely or in back box.

- Five- year parts and one year labor limited warranty\*
- Individual 2.85" LCD touch screen display for each gas service
- The LCD touch screen allows all alarm programming and set up to be done without the use of tools
- A green normal or red alarm condition for each gas service confirms the condition for each individual gas service
- Emergency preparedness instructions Med Touch Alarm Panels allow users to set up customized instructions for each alarm signal which appear when the signal is in alarm
- Up to 4 area/gas boards in a two slot box, 8 in a three slot and 16 in a five slot panel
- Last event history (per gas board /signal point)
- Optional Ethernet board which provides; alarm notification to up to 5 addresses, website with real time graphic alarm face image, event log of last 100 events and Modbus interface with building management system.
- Optional BACnet module which provides BACnet interface of alarm conditions with building management system.
- Our Med Touch "T" Series of alarms may be interfaced with Lon works using our 35-3032 Ethernet/ Rabbit board which provides a Modbus interface. Modbus provides interface with Lon works.
- Manufactured & Tested in the U.S.A. with Domestic & Imported Components.
- NFPA and ISO pre-loaded gas 'labels'
- English, English/French and English/ Spanish pre-loaded languages
- Editable text and alarm 'labels' .
- Self-contained unit designed for ease of installation & service
- Self-diagnostic error message display for ease of maintenance
- Audio and visual alarm indicators
- Bright easy to read LCD displays clearly visible in both day and night lighting conditions
- Constant display and monitoring of each gas
- User programmable high/low set points
- Dry contacts for remote monitoring of all alarm conditions on each gas module and on the CPU module for the entire panel
- Easy to read color coded gas modules
- Hinged frame for easy accessibility
- Color coded displays

In addition, each Area Alarm Module shall incorporate the following features:

- Does not require re-calibration •
- Gas specific sensor with DISS nut & nipple. An error message will be displayed if incorrect sensor or no sensor is attached
- Shall be capable of displaying gas readouts in PSI, kPa, BAR, inHg, or mmHg
- Gas alarm repeat feature is factory set at 10 minutes, and is adjustable or may be turned off

\* See Terms and Conditions, Document No. 99-0477, on our website at: www.tri-techmedical.com for complete details.



### Specification

The alarm shall be the Tri-Tech Medical Med Touch Area Alarm. The panel shall be microprocessor controlled and designed to comply with NFPA 99. The panel shall incorporate an individual 2.85" LCD touch screen display for each gas service which shall allow all alarm programming and set up to be done without the use of tools. The panel shall be 100% digital and shall not require re-calibration. The unit shall be enclosed in a steel box and shall be designed to accept an electrical input range of 120-240 volts AC - 50-60 hertz. The source voltage shall be stepped down with a self-contained transformer. The panel shall contain audible and visual alarm indicators. The audible alarm may be silenced by pressing the alarm silence button, but the visual alarm indicator can only be cancelled by fault correction. The alarm shall detect and filter out transient (less than 0.6 seconds) signals created by R.F.I. The alarm shall be capable of displaying alarm history for all possible alarm conditions. The alarm shall be capable of monitoring and displaying up to 16 gases per alarm panel. Gas modules can be arranged in accordance with the customer's requirements. A security pin code shall be required and must be entered in order to access the programming software.

e7248rH 05/20/24



The following options shall be able to be added to the panel at the time of order or after installation;

- Optional Text / e mail notification of alarm events sent to up to 5 addresses
- Optional Ethernet module. This module may be added to any alarm panel. It will provide the following features:
  - Webserver allows a remote user access to the alarm's webpage viewing a graphic image of the alarm with all signal conditions, pressure & event / history log information
  - Text notification of alarm conditions (up to 5 addresses)
  - Event / history log maintains a rolling list of the 100 most recent alarm condition events and a file of the 1,000 most recent alarm events and is downloadable through embedded web page
  - Modbus interface
- Event log is viewable and downloadable through embedded web page, including the most recent 1,000 events.
- Optional Ethernet connectivity with embedded web page
- Interface with building automation system via Ethernet







# HIGH Low Gas REPEAT CAL UNITS O TEXT SET COMID TEXT SET BACK INPUT VIEW



# Gas / Area Display

Shown in alarm condition displaying rooms/area monitored. Display blinks on and off when in alarm condition drawing attention and alternates displaying rooms/area monitored with user programmed emergency instructions.

Shown in alarm condition displaying emergency instructions. Display blinks on and off when in alarm condition drawing attention and alternates displaying rooms/area monitored with user programmed emergency instructions.

Shown in normal condition. Code specific gas colors, and gas name are displayed at the top in facilities language. Gas pressure and units of measure are displayed in the middle of the screen. Normal or Alarm condition is displayed in the bottom center. Rooms/area monitored alternates with user programmed emergency instructions at the bottom.

Easy to use icon menu provides quick access to programming options.

Security is built in to each circuit board. The correct four digit pin code must be entered in order to access the programming software.

e7248rH 05/20/24

Tri-Tech Medical Inc., 35401 Avor


# How to Order: Easy to use modular ordering system.

**Tri-Tech** 

**Medical Inc.** 

Fill in the blanks to specify the Med Touch Area Alarm that meets your needs.

# Model Numbering:

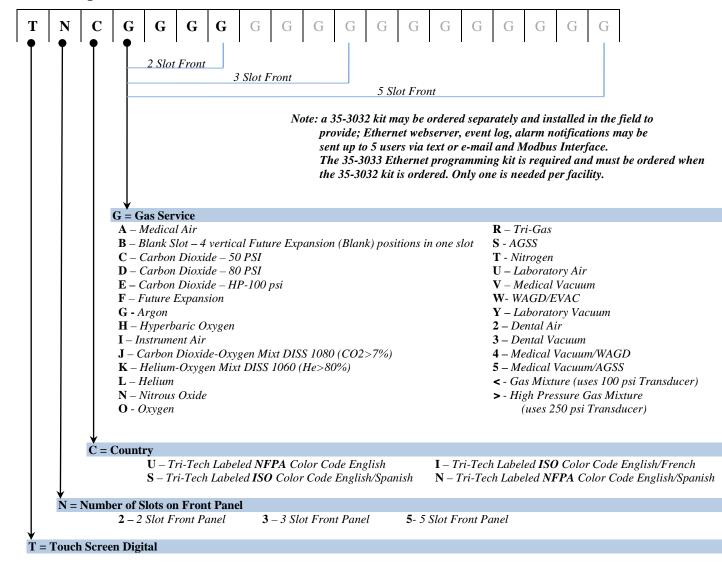



#### Part No. Examples

\




$$\label{eq:transformed} \begin{split} \textbf{T2UOVAF} &= 2 \text{ Slot Area Alarm Tri-Tech labeled, set for NFPA Color Code English. Gas set Oxygen,} \\ & \text{Medical Vacuum, Medical Air gas modules in $2^{nd}$ slot. Last module in $2^{nd}$ slot is for Future Expansion (Blank).} \end{split}$$



 T3IOVANTSFF2 = 3 Slot Area Alarm Tri-Tech labeled, set for ISO Color Code English/French. Gas set

 Oxygen, Medical Vacuum, Medical Air and Nitrous Oxide in 2<sup>nd</sup> slot, Nitrogen,

 AGSS and two Future Expansion (Blank) modules in 3<sup>rd</sup> slot.

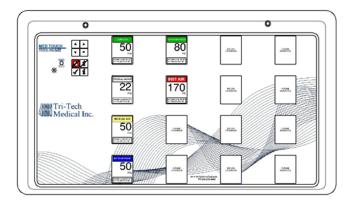

e7248rH 05/20/24

T5UOVANHIFFBB2 = 5 Slot Area Alarm Tri-Tech labeled, set for NFPA Color Code English. Gas set Oxygen, Medical Vacuum, Medical Air and Nitrous Oxide in 2<sup>nd</sup> slot, Hyperbaric Oxygen, Instrument Air and two Future Expansion (Blank) positions in 3<sup>rd</sup> slot (Slots 4 and 5 are Blank).

No. 99-0489



# Tri-Tech Medical Inc.

