

Retro-Fit Manual Diamond 3 Outlet



Important Note

Due to unknown factors of the existing equipment, projects requiring Retro-Fit Conversions must be fully investigated by the installer.

It is the responsibility of the installer to determine if the Retro-Fit will function correctly and if additional components are required. Amico is not responsible for any additional components required.


Retro-Fit Instructions

This Retro-Fit should only take about 10 minutes to install. There is no need to shut off the gas. Please note that this application only Retro-Fits the front assembly and no parts will be sent along for the back rough-in. If you require a Retro-Fit to change the back rough-in you'll need to order the Diamond 2 Retro-Fit.


Ohmeda Diamond 2 & 3 Outlet
Remove screws from front Facia.



Amico has made this latch valve compatible to the existing rough-in. The barrel is shorter and the diameter is wider. Note the indexing of the pin is on the bottom.

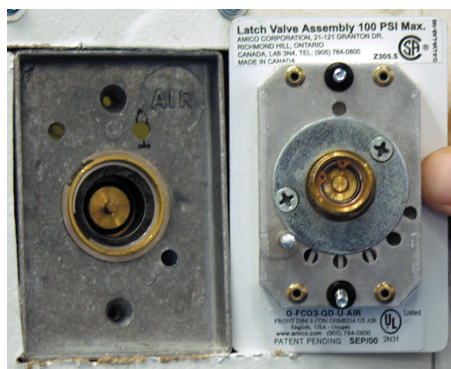

Front View

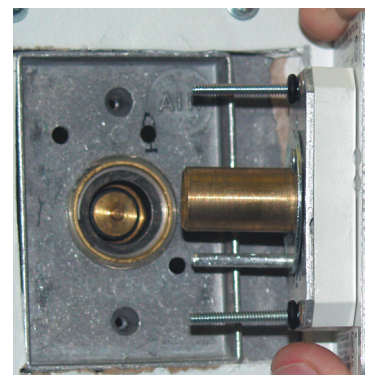

Side View

Retro-Fit Instructions

Screw in the Amico Diamond 3 Latch Valve and 4.5" chrome frame.



A completed Diamond 3 assembly conversion from Ohmeda to Amico.



WARNING

Ohmeda issued a Mandatory Field modification for its back body on Diamond 3 Outlets manufactured between 1990 and 1993. If this Ohmeda field modification was not applied, the Outlet **MUST NOT** be Retrofitted since this could cause total loss of gas flow through the Outlet.

www.amico.com

Amico Pipeline | www.amico.com

85 Fulton Way, Richmond Hill
Ontario, L4B 2N4, Canada

71 East Industry Court, Deer Park
NY 11729, U.S.A

Toll Free Tel: 1.877.462.6426

Toll Free Fax: 1.866.440.4986

Tel: 905.764.0800

Fax: 905.764.0862

Email: info@amico.com